Duplica^{RealTime} *M. tuberculosis* Complex Kit

REV: EBR019032_IFU_REV.04D_ENITA

REF: EBR019032 -32 tests

Instructions For Use

INTENDED USE

Duplica^{RealTime} *Mycobacterium tuberculosis* Complex Kit is an *in vitro* nucleic acid amplification test for the detection of *Mycobacterium tuberculosis* complex (*M. tuberculosis*, *M. africanum*, *M. bovis* and *M. microti*) in clinical specimens. The system is validated for the following clinical samples: Pulmonary Samples (BAL, sputum, endotracheal aspirate); Body Fluids (cerebrospinal fluid, seminal fluid, urine, pleural fluid); Biopsy (liver, femoral biopsy, placenta); Lymphonodes; Pus; Stool.

INTRODUCTION

Tuberculosis (TB) is caused by gram-positive bacteria known as the *Mycobacterium tuberculosis* complex (MTBC). MTBC includes the typical human-associated pathogens *M. tuberculosis* and *M. africanum*; *M. canettii* and other so-called "smooth TB bacilli", the actual host range of which remains unknown; and several lineages adapted to different mammal species that include *M. bovis*, *M. microti*, *M. caprae*, *M. orygis*, and *M. pinnipedii* (4–6). Because of the wider host range of animal-associated MTBC, the common view until a decade ago was that human TB strains had evolved from *M. bovis*, the typical agent of bovine TB. Recent comparative genomic analyses have challenged this view by showing that animal MTBC strains nest within the genetically more diverse human MTBC strains. These results not only contradict the hypothesis of an animal origin for human MTBC but also promote an alternative scenario for a human origin of animal MTBC.

PRINCIPLE OF THE TEST

Duplica^{RealTime} *M. tuberculosis* Complex Detection Kit is an assay for the DNA-based detection of the *Mycobacterium tuberculosis*, *Mycobacterium africanum*, *Mycobacterium bovis* and *Mycobacterium microti*. The kit is designed to amplify a portion of 16S-23S rRNA gene sequences. The reagents for the amplification are ready-to-use and provided in 3 separate tubes:

- **AMPLIFICATION MIX:** with Hot Start Taq DNA polymerase, nucleotides, MgCl₂ and buffer.
- **OLIGO MIX:** with primers and fluorogenic probes.
- **INTERNAL CONTROL OF AMPLIFICATION:** to check the successful of amplification reaction.

Duplica^{RealTime} *M. tuberculosis* Complex Detection Kit is based on specific recognition and amplification of target sequences by PCR, and the simultaneous detection of the accumulation of PCR amplification products by fluorescent DNA probes. The probe designed to detect the target carries the fluorophore FAM (6-carboxy-fluorescein) at the 5' end, while the probe detecting the internal control (IC), is labelled with the fluorophore HEX (hexa-chloro-fluorescein). Both the probes have a non fluorescent quencher at the 3' end. If excited, the whole probe doesn't emit fluorescence, since the proximity of the quencher to the reporter prevents the emission of the fluorescence from the reporter (quenching effect).

REAGENTS PROVIDED

Each kit contains enough reagents to perform 32 tests when used in 4 analytical sessions with 6 **samples**, 1 **Positive Control of Amplification (Control 1, C1)**, and 1 **Reaction Blank (BM)** each.

Kit Components

Reagent	Color Code	Storage (range, °C)	Volume (µl)	Quantity (tubes)
Oligo Mix (OM)*	Green Cap	-22÷-18	200	1
Amplification Mix (AM)	Blue Cap	-22÷-18	400	1
Control 1 (C1, Positive C.)	Red Cap	-22÷-18	>50	1
Control 2 (C2, Internal C.)	Yellow Cap	-22÷-18	>50	1
Reaction Blank (BM)	Neutral Cap	-22÷-18	>50	1

*protect the tube from direct light

STORAGE AND HANDLING

All reagents must be stored at **-22÷-18°C** and can be used until the expiry date printed on the labels. Do not freeze and thaw the products more than six times.

MATERIAL REQUIRED BUT NOT PROVIDED

- Extraction kit for DNA purification (refer to the specific handbook's section)
- Optical tubes or microplate for Real Time PCR
- Disposable powder-free gloves and laboratory coat
- Variable volumes pipettes (5-20 µl, 20-200 µl and 100-1000 µl)
- Disposable RNase/DNase-free tips with aerosol barriers
- Tube racks
- Desktop centrifuge
- PCR box
- Refrigerator
- Deep freezer
- Thermal cycler for Real Time PCR

The kit has been developed to be used on SmartCycler® (Cepheid), Rotor-Gene® Q (Qiagen) and DX®/CFX (Bio-Rad) thermal cyclers. Other makes and models should be fully tested and evaluated for optimal performance by the user before reporting results.

The equipment should be regularly maintained, in accordance with the manufacturer's instructions, and calibrated to ensure accurate PCR cycling and optimal performance.

PRECAUTIONS AND WARNINGS

- In compliance with Good Laboratory Practice, define three separate laboratory's areas for: DNA extraction, PCR reaction mix preparation; manipulation of controls provided with the kit. Each area must have dedicated pipettes and laminar flow hood.
- If required, Clonit offers the necessary technical support for the correct use of the kit.
- Carefully read this Instruction for Use before using the kit
- Do not use the reagents after the expiry date
- Thaw and carefully mix the reagents of the kit before use
- Do not mix the reagents from different lots of the product
- Use calibrated and regularly checked pipettes and instrumentation only
- Use dedicated laboratory equipments. Change gloves frequently
- Periodically wipe the working area with 0,5% hypochlorite
- Use powder-free gloves. Do not leave fingerprints on optical section of Smart Tubes.
- Materials containing or potentially containing infectious agents must always be manipulated in a separated microbiological safety room under a Biohazard biological hood.
- In case of damaged package, contact the technical support before using the kit
- Do not use the product when stored at temperatures other than those indicated on the labels or described in this Instructions For Use.
- In case of spillage of the kit contents, please refer to the specific Material Safety Data Sheet (MSDS, available on request).
- The kit reagents, individual protective equipments, used materials, biological samples and test residuals must be disposed in accordance with local regulations.
- Patient Drug treatment may interfere with the final result of the molecular biology analysis.

OPERATING PROCEDURE

a) DNA purification

For genomic DNA purification Clonit recommends to use:

- Duplica®Prep NA Body Fluid kit (ref. EDI004200) with Duplica® PREP Automatic Extractor (ref. EDI001).
- Bact Extra Pure Kit (ref. EDR004050) for manual extraction

Other extraction reagents and methods should be fully tested and evaluated for optimal performance by the user before reporting results.

b) Thermalcycler Setup

Important points before starting: Refer to the specific handbook of the equipment used to set the thermal profile indicated in the **Thermal Profile Table**. We recommend to switch on the instrument and to set the thermal profile before preparing the reaction mix.

N.B.: before starting the run is recommended to save the file as "M. tuberculosis Complex Test"; this way it is possible to save the Thermal Profile and settings and recall them in subsequent runs.

SmartCycler® platform

- Open the SmartCycler® software and click "Define Protocols".
- Click "New Protocol" (bottom left).
- Enter Name " **M. tuberculosis Complex Test** ".
- Enter the parameters for the **M. tuberculosis Complex Test** outlined in Table below
- When ready click "Save Protocol".

Rotor-Gene® Q platform

- Start the software and on the box *New Run* select *Advanced*
- Select a *new template* in *Empty Run* or a pre-existing one
- Select the Rotor Type of your instrument and then *Next*
- Type 25 µl in the reaction volume and then *Next*
- Select *Edit Profile* and set up the correct Thermal Profile as indicated in the table below
- Select *Gain Optimisation* and then flag the option *Perform Optimisation before 1st acquisition*
- On *Channel Settings* select the green/yellow fluorophores and tube position "1" to perform the optimization. Then close the window and select *Next* and *Start Run*

DX®/CFX platform

- Select *Create a new Experiment*
- In the *Protocol* section select *Create New* to set the thermal profile
- In *Plate Editor*: set FAM and as *Fluorophores* and define the sample name.

Thermal Profile Table (common for all the supported platforms)

TIME	TEMPERATURE	CYCLES
10 min	95°C	1
15 sec	95°C	55
60 sec	62°C	Fluorescence Acquisition

c) PCR mix preparation

The total reaction volume is **25 µl**.

For each experiment prepare a PCR mix for the positive control (**C1**), the **Reaction Blank (BM)** and **n+1** samples. The internal control of amplification (**C2**) should be added directly to the PCR MIX, prior the aliquotation. The reagents of the PCR mix have to be mixed as indicated in the table below:

REAGENT	VOLUME (µl)
Amplification mix	10
Oligo mix	5
Internal Control (C2)	1
Extracted DNA	9

The PCR mix has to be freshly prepared every time

After its preparation, aliquot **16 µl of Master Mix** in the tubes or in the microplates well for PCR then add in each tube/well **9 µl** from the **extracted DNA** or **Control 1 DNA**, place in order the tubes/microplate in the instrument and start the program of amplification. At the end of the program remove the tubes/microplate from the thermalcycler.

d) ANALYSIS and INTERPRETATION of RESULTS

Important points before starting: For a detailed description on how to analyze data, refer to *System User's manuals*. **Always visually inspect the amplification plot for each sample tested versus C_T values obtained with the software.**

Results interpretation

Refer to the instrument-specific user guide to visualize the amplification plots for the entire plate/rotor. Detailed analysis of raw data depends on the real-time PCR instrument used. Baseline noise levels should either be set automatically or at predefined cycles.

The fluorescence in each channel indicates the hybridisation of the target specific probes:

- **Channel 1** for **FAM/Green= Target probe**
- **Channel 2** for **HEX/Cy3/Yellow= Internal Control probe**.
- If a sample shows a fluorescence in **FAM/Green (C_T>0)**, the sample is surely positive and the signal detected by **HEX/Cy3/Yellow** fluorophore (**C_T≥0**) is not relevant.
- When no signal at fluorophore **FAM/Green (C_T=0)** is detected, to confirm the negative result, the completed amplification of internal control and therefore the appearance of a fluorescent signal at **HEX/Cy3/Yellow (C_T>0)** fluorophore level, must be verified. Only in this case we can state that the sample is definitely negative.
- Condition in which no signal is detected indicates PCR inhibition. The sample must to be repeated and a dilution 1:10 of the target DNA is suggested.

Please be aware that:

- The Blank is positive in the **HEX/Cy3/Yellow** channel, but negative in the **FAM/Green** channel.
- The Positive Control is positive in both the **FAM/Green** and the **HEX/Cy3/Yellow** channels. The **HEX/Cy3/Yellow** signal, due to competition with **FAM/Green** probe, may arise later or be absent. In this case the **HEX/Cy3/Yellow** signal is not relevant and the run is valid.

If all these conditions have been met, the run is valid and it's possible to analyse the data; otherwise the run is not valid. It's responsibility of the user to validate the run.

Results Interpretation Table

FAM/Green	HEX/CY3/Yellow	Results
$C_T > 0$	Not relevant ($C_T \geq 0$)	POSITIVE
$C_T = 0$	$C_T > 0$	NEGATIVE
$C_T = 0$	$C_T = 0$	INHIBITION

TROUBLESHOOTING

Problem 1: Weak or no signal of Positive Control, C1.

1. The PCR conditions did not comply with the instructions. All sample results are INVALID:
 - Check the amplification protocol and select the fluorescence channel reported in the manual.
2. Deterioration of dyes and/or primers. The reagents storage conditions did not comply with the instructions:
 - Check storage conditions.

Problem 2: Weak or no signal of the Internal Control, C2 in unknown samples AND Reaction Blank BM.

1. The PCR was inhibited:
 - Make sure to use a recommended DNA purification method and carefully follow the manufacturer's instructions.
2. Pipetting error due to omitted reagents or samples:
 - Repeat the analysis starting from the PCR.
3. Deterioration of dyes and/or primers. The reagents storage conditions did not comply with the instructions:
 - Check storage conditions.
4. Very low starting amount and/or low purity of genomic DNA. Improper DNA extraction:
 - Repeat the DNA purification.
5. Wrong channel/filter was chosen. The PCR conditions did not comply with the instructions:
 - Check the PCR conditions and select the fluorescence channels reported in the protocol for the Unknown Sample detection.

Problem 3: FAM signal in Reaction Blank BM.

1. Contamination during PCR preparation procedure. All samples results are INVALID:
 - Decontaminate the working area and all instruments.
 - Pipette the controls C1 and C2 at last.
 - Repeat the PCR preparation using a new set of reagents.

Problem 4: No FAM and HEX signals in unknown sample.

1. The PCR was inhibited:
 - Make sure to use a recommended DNA purification method and carefully follow the manufacturer's instructions.

Problem 5: Wide Fluctuations in fluorescence values.

1. The PCR Master Mix is not well prepared:
 - Carefully repeat the PCR preparation procedure.
2. Air bubbles trapped in the PCR tubes:
 - Check the presence of air bubbles before starting a new run.

Problem 6: Absence of any fluorescent signal.

1. Verify the performance of the thermal cycler:
 - Calibrate the equipment.
2. Deterioration of dyes and/or primers. The storage conditions did not comply with the instructions:
 - Check storage conditions.
 - Check the expiry date of the kit.

Problem 7: The thermal cycler gives an error message.

1. Refer to the real-time PCR instrument user manual or contact the local technical support of the real-time PCR instrument company.

Problem 8: The kit reagents left out of the storage range temperature.

1. These reagents must be stored **as indicated** for a proper execution of the test. The performance of the product is not guaranteed if the reagents have not been properly stored.

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CLONIT S.r.l.