



SZABO SCANDIC

Part of Europa Biosite

Produktinformation



Forschungsprodukte & Biochemikalien



Zellkultur & Verbrauchsmaterial



Diagnostik & molekulare Diagnostik



Laborgeräte & Service

Weitere Information auf den folgenden Seiten!
See the following pages for more information!



Lieferung & Zahlungsart

siehe unsere [Liefer- und Versandbedingungen](#)

Zuschläge

- Mindermengenzuschlag
- Trockeneiszuschlag
- Gefahrgutzuschlag
- Expressversand

SZABO-SCANDIC HandelsgmbH

Quellenstraße 110, A-1100 Wien

T. +43(0)1 489 3961-0

F. +43(0)1 489 3961-7

mail@szabo-scandic.com

www.szabo-scandic.com

[linkedin.com/company/szaboscandic](https://www.linkedin.com/company/szaboscandic) 

Light Green Solution (0.1%)

Description: Light Green Solution (0.1%) is intended for use in histological applications as a counterstain for Grocott's Silver Stain in addition to other procedures. When used correctly, various shades of green can be obtained to aid in visualization of tissue components.

Cytoplasm:	Light Green
Nuclei:	Light Green/Blue

Uses/Limitations: Not to be taken internally.
For In-Vitro Diagnostic use only.
Histological applications.
Do not use past expiration date.
Use caution when handling reagents.
Non-Sterile.

Availability:	<u>Item #</u>	<u>Volume</u>
	LGD125	125 ml
	LGD500	500 ml
	LGD999	1000 ml


Storage: Store at 18-25°C.

Precautions: Avoid contact with skin and eyes.
Harmful if swallowed.
Follow all Federal, State, and local regulations regarding disposal.

Procedure:

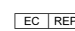
1. Deparaffinize sections if necessary and hydrate to distilled water.
2. Rinse slide in distilled water.
3. Stain slide in Light Green Solution (0.1%) for 1-2 minutes.
4. Rinse 1 time in Distilled/Deionized water.
5. Dehydrate quickly through graded alcohols.
6. Clear, and mount in synthetic resin.

Storage: 18° C  25° C



ScyTek Laboratories, Inc.
205 South 600 West
Logan, UT 84321
U.S.A.

 EmergoEurope (31)(0) 70 345-8570
Molsnstraat 15
2513 BH Hague, The Netherlands